

- 1. Insert the Euro FM receiver into the three pin socket on the Euro Accessory Adaptor.
- 2. Turn on the Euro FM receiver by pushing the latch to the on position (2 dots).
- 3. Turn ON your processor.
- 4. Connect the Euro Accessory Adaptor or the MI14i to the processor accessory socket.
- Return processor to user.

Testing the radio aid with the listening headphones:

- 5. Insert the Euro accessory adaptor into the socket in the middle of the headphones.
- 3. Turn ON the processor.
- 4. Connect the headphones to the processor accessory socket.
- 6. Listen to the earphones to check sound quality.



Once the student has the radio aid in place check that they can hear clearly through the radio aid.