





## Nucleus® Freedom™ Bodyworn Speech Processor Quick Reference Guide

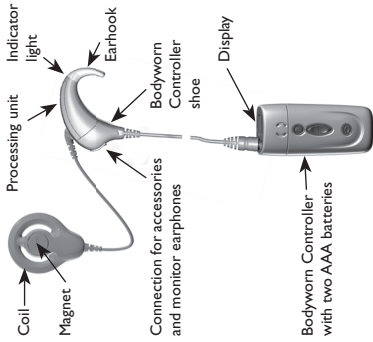

### Parts of your speech processor

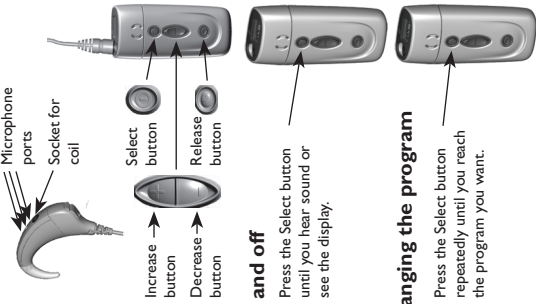

### Changing microphone sensitivity

Press the Increase or Decrease buttons until you reach the level you want.




### Changing batteries


1. Press.



2. Slide the cover down.

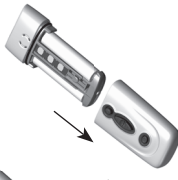

3. Change the batteries, making sure the + and - symbols on the batteries match the + and - symbols in the compartment.

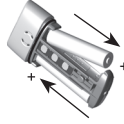

4. Slide the cover back until it clips into place.

Program	Description of the program and/or when to use the program
P1	
P2	
P3	
P4	

Magnet strength:  
Transmitting cable length:



Cochlear Ltd (A/NZ) 6, 103/118  
 Tel: 61 3 9292 8533 Fax: 61 3 9292 8332  
 www.cochlear.com

Cochlear Americas  
 499 Inverness Parkway  
 Suite 400  
 Englewood, CO 80113  
 Tel: 1 303 790 9033  
 Fax: 1 303 790 8425

Cochlear AG  
 Europastrasse 47  
 D-18053 Hohenlocke  
 Switzerland  
 Tel: 41 205 9454  
 Fax: 41 205 9454

Cochlear Australia  
 218 Burwood Drive  
 Burwood, NSW 1585  
 Australia  
 Tel: 61 2 9511 4270  
 Fax: 61 2 9511 4270

Cochlear Canada Ltd  
 24-30 Inverness Street  
 1100 Toronto  
 Ontario, M6R 2K9  
 Canada  
 Tel: 416 291 9600  
 Fax: 416 291 9600

Cochlear Europe  
 24-30 Inverness Street  
 1100 Toronto  
 Ontario, M6R 2K9  
 Canada  
 Tel: 416 291 9600  
 Fax: 416 291 9600

Cochlear France  
 24-30 Inverness Street  
 1100 Toronto  
 Ontario, M6R 2K9  
 Canada  
 Tel: 416 291 9600  
 Fax: 416 291 9600

Cochlear Germany  
 Europastrasse 47  
 D-18053 Hohenlocke  
 Switzerland  
 Tel: 41 205 9454  
 Fax: 41 205 9454

Cochlear Hong Kong  
 24-30 Inverness Street  
 1100 Toronto  
 Ontario, M6R 2K9  
 Canada  
 Tel: 416 291 9600  
 Fax: 416 291 9600

Cochlear India  
 24-30 Inverness Street  
 1100 Toronto  
 Ontario, M6R 2K9  
 Canada  
 Tel: 416 291 9600  
 Fax: 416 291 9600

Cochlear Japan  
 24-30 Inverness Street  
 1100 Toronto  
 Ontario, M6R 2K9  
 Canada  
 Tel: 416 291 9600  
 Fax: 416 291 9600

Cochlear Korea  
 24-30 Inverness Street  
 1100 Toronto  
 Ontario, M6R 2K9  
 Canada  
 Tel: 416 291 9600  
 Fax: 416 291 9600

Cochlear Mexico  
 24-30 Inverness Street  
 1100 Toronto  
 Ontario, M6R 2K9  
 Canada  
 Tel: 416 291 9600  
 Fax: 416 291 9600

Cochlear New Zealand  
 24-30 Inverness Street  
 1100 Toronto  
 Ontario, M6R 2K9  
 Canada  
 Tel: 416 291 9600  
 Fax: 416 291 9600

Cochlear Norway  
 24-30 Inverness Street  
 1100 Toronto  
 Ontario, M6R 2K9  
 Canada  
 Tel: 416 291 9600  
 Fax: 416 291 9600

Cochlear Spain  
 24-30 Inverness Street  
 1100 Toronto  
 Ontario, M6R 2K9  
 Canada  
 Tel: 416 291 9600  
 Fax: 416 291 9600

Cochlear Sweden  
 24-30 Inverness Street  
 1100 Toronto  
 Ontario, M6R 2K9  
 Canada  
 Tel: 416 291 9600  
 Fax: 416 291 9600

Cochlear Switzerland  
 24-30 Inverness Street  
 1100 Toronto  
 Ontario, M6R 2K9  
 Canada  
 Tel: 416 291 9600  
 Fax: 416 291 9600

Cochlear Taiwan  
 24-30 Inverness Street  
 1100 Toronto  
 Ontario, M6R 2K9  
 Canada  
 Tel: 416 291 9600  
 Fax: 416 291 9600

Cochlear Thailand  
 24-30 Inverness Street  
 1100 Toronto  
 Ontario, M6R 2K9  
 Canada  
 Tel: 416 291 9600  
 Fax: 416 291 9600

Cochlear USA  
 499 Inverness Parkway  
 Suite 400  
 Englewood, CO 80113  
 Tel: 1 303 790 9033  
 Fax: 1 303 790 8425

Cochlear Vietnam  
 24-30 Inverness Street  
 1100 Toronto  
 Ontario, M6R 2K9  
 Canada  
 Tel: 416 291 9600  
 Fax: 416 291 9600


Cochlear UK  
 24-30 Inverness Street  
 1100 Toronto  
 Ontario, M6R 2K9  
 Canada  
 Tel: 416 291 9600  
 Fax: 416 291 9600

Nucleus is a registered trademark of Cochlear Limited.  
 Cochlear, the Cochlear logo and Freedom are  
 trademarks of Cochlear Limited.  
 © Cochlear Limited 2005  
 Printed in Australia  
 N30863F ISS1




## Using an accessory and/or the monitor earphones

Accessories:

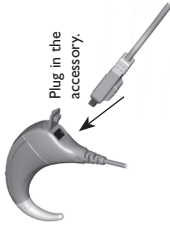

Accessories and earphones:

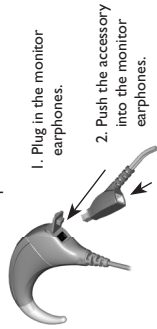

1. Plug in the monitor earphones.

2. Push the accessory into the monitor earphones.


## Lock and unlock

Press the Select button and the Decrease button to lock or unlock the processor.



## Changing volume

1. Briefly push either the Increase or Decrease buttons.
2. Push again, for a slightly longer time, on either the Increase or Decrease buttons until you hear a beep and/or see a flashing 'V' on the display.
3. Continue briefly pushing either the Increase or Decrease buttons until you reach the volume level you want.

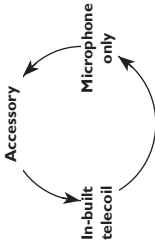

## Using the in-built telecoil

Push both the Increase and Decrease buttons until 'T' shows on the display.



## Resetting the speech processor

Press all three buttons (Select, Increase and Decrease) together (for at least one second) to return to the default settings.



## Mic lock

Attach the microphone lock as shown.



Help message	Meaning	Action
H1	Flat battery	Change all batteries. Note: you will stop hearing sound and only be able to turn off the speech processor.
H2	Low battery	Change all batteries.
H3	Coil error	Check: <ul style="list-style-type: none"> <li>• coil position</li> <li>• cable connections</li> <li>• for cable damage.</li> </ul>
H4	Audio error	Follow the steps in the Troubleshooting section of your User Manual to try to resolve the problem.
H5	Program corrupted	Use an alternate program where it is available until you are able to return the speech processor to your clinician. They can install a new program for you.

